

ECLIPSE®

HEAT & SEAT™

Resilient Lined Nightguards

Professional Information

Insertion:

Flow hot tap water (130°F/54°C) over the intaglio surface of the appliance for 20-30 seconds, then seat in the patient's mouth.

Removal:

Ask patient to rinse mouth with warm water for 5-10 seconds, then carefully remove.

Disinfecting:

The following disinfectants have been found to work well with cured Eclipse Resins: ProSpray, Cavicide, Banicide, Prophene™ Plus, and Birex®se.

ProSpray™, Cavicide®, Banicide®, Prophene™ Plus, and Birex®se are not products of DENTSPLY International.

www.trubyte.dentsply.com or www.dentsply.com

Dental Professional: Tear here at perforation and provide other half to patient.

ECLIPSE®

HEAT & SEAT™

Resilient Lined Nightguards

Patient Information

Insertion:

Flow hot tap water (130°F/54°C) for 20-30 seconds over the surface of the appliance that contacts teeth.

Removal:

Rinse your mouth with warm water for 5-10 seconds and carefully remove.

Cleaning:

You may lightly brush your nightguard with a soft toothbrush and toothpaste.

Your Eclipse Heat & Seat Nightguard may also be cleaned with common denture cleaners including: Polident® Overnight, Efferdent® Plus and Fixodent®.

Polident Overnight, Efferdent Plus and Fixodent are not trademarks of DENTSPLY International, Inc.

DENTSPLY Trubyte, DENTSPLY International Inc. York, PA 17405-0872

www.denturehelp.com

905472 Rev. A 4326-H (5/06)