

Lucitone® FRS® Flexible Dental Resin

Professional Instructions

This high-quality dental appliance is made from Lucitone® FRS® Flexible Dental Resin. Your dental laboratory has injected this nylon material for a precise fit. If adjustments are needed, please use Faskut® Luci-Sof® Burs. These straight handpiece burs are available in *small* (order #904763), *medium* (#904764) and *large* (#904765). Polish with normal procedures used for denture acrylics.

Dental Professional: Tear here at perforation and provide other half to patient.

Lucitone® FRS® Flexible Dental Resin Patient Instructions

Congratulations, your dental professional has prescribed a comfortable and flexible dental appliance made from Lucitone® FRS® nylon dental resin. A critical factor in longevity of this appliance is how you care for it.

Clean your appliance regularly with a commercial denture cleanser, at least 2-3 times per week is recommended. You may use effervescent cleaning solutions such as *Efferdent® Denture Cleanser*. You should also use a denture cleansing toothpaste or low abrasive toothpaste with a soft toothbrush or denture brush at least daily.

The denture is very tough and strong but is not unbreakable. Be careful not to drop the appliance on hard floors. Avoid excessive flexing.

Efferdent is not a trademark of DENTSPLY International Inc.

DENTSPLY Trubyte, DENTSPLY International Inc., 570 West College Avenue, York, PA 17405-0872
www.trubyte.dentsply.com or www.dentsply.com Form No. 4236-M (8/10) 904881 Rev. C