



E4D - PlanScan

To Export STL Files:

Doctors with an E4D system can send digital impressions directly to Mabel Dental Lab through E4D Sky.

1. Setup, scan, and mark your margin. Click 'E4D SKY' on the top right hand corner.
2. Click 'Export' and Send Buccal Bite Case.
3. Before you click 'Submit' make sure the boxes are selected to save and email the STL files.
4. Once you click 'Submit' a new screen asking to upload the STL files will appear. Select your prep, opposing, or buccal bites.
5. When you click 'Browse', a folder containing the STL files will appear, then select the coordinating files.
6. Once all files have been uploaded, enter your email address and Mabel Dental Lab email: support@mabeldental.com
7. **IMPORTANT NOTE: Do not close the E4D software or turn off the system until the case has been fully transferred.**

The file is now saved in the location as directed by the user and can be copied to a flash drive or uploaded to a file sharing service as needed. Mabel Dental Lab recommends sending us your files via the website: WeTransfer.com. Please see the next page for instructions.


To upload the digital impression to Mabel Dental Lab, we recommend using www.wetransfer.com.

1. Go to WeTransfer.com.
2. Choose the "Take Me to Free" option.
3. Choose "I Agree" to the Terms and Condition
4. Click "+ Add your files" and select the files from your computer that you want to transfer.
5. In the "Email to" field, enter support@mabeldental.com
6. Enter your email address in the "Your email" field
7. Enter an optional message
8. Click the "Transfer" button

Voila! The file(s) has been sent!

We recommend that you fax/email a Prescription Rx as well.

You'll receive confirmation within 1 business day.

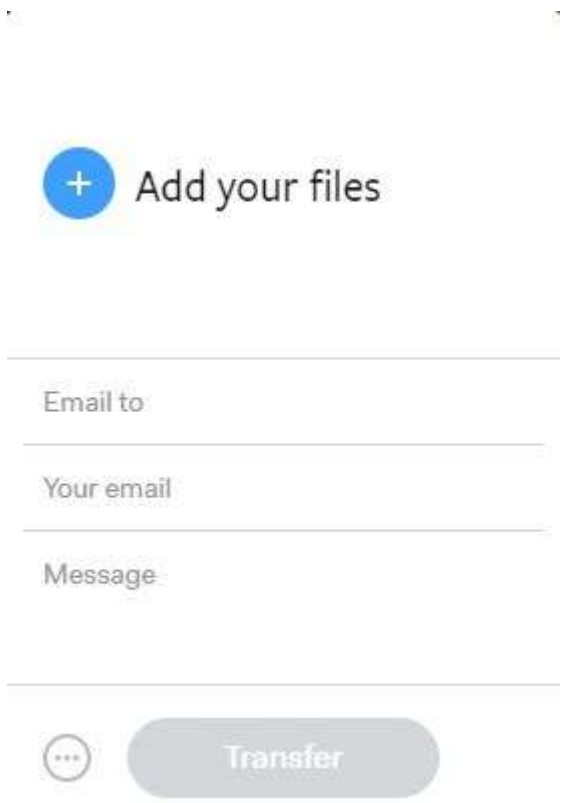

+ Add your files

Email to

Your email

Message



Transfer

*please note this may not reflect the exact look of the screen